

Video: CapTel 880iB

Transcript

“CapTel ® 880iB” and “SETTING UP WITH HUMANWARE BRAILLIANT ®” are shown on the screen.

Zooms out from CapTel 880iB phone located on a counter next to a Humanware Brailiant display.

Narrator: “The CapTel 880iB can connect with the Humanware Brailiant 40-cell Braille display for Braille captions.”

Narrator: “Make sure your CapTel 880iB is set up, plugged in, and powered on where you would like to use it.”

Two men in a kitchen. One is sitting and the other is standing. The man sitting turns on the Brailiant display.

Narrator: “To connect, first turn on your Brailiant.”

The man standing pressed YES button on the CapTel 880iB.

Narrator: “Then, with the handset hung up, press the YES button on the CapTel 880i phone to see the options menu.”

The man standing pressed DOWN arrow button and scrolls down on the CapTel 880iB option screen.

Narrator: "Press the DOWN arrow button to highlight braille display settings."

Close-up of CapTel 880iB screen as the man standing continues to press the buttons that the narrator speaks about each time.

Narrator: "Press the YES button to select."

Narrator: "Press the DOWN arrow button to highlight Bluetooth options."

Narrator: "Press the YES button to select."

Narrator: "Press the DOWN arrow button to highlight View Bluetooth Device to make sure nothing else is paired with your CapTel phone."

Narrator: "Press the YES button to select. The screen should read No Device Paired."

Narrator: "Press the NO button to go back to the Bluetooth settings menu."

Narrator: "Press the DOWN arrow button to highlight Scan for Bluetooth Devices."

Narrator: "Press the YES button to select."

The man sitting and the man standing wait for the phone to complete process.

Narrator: "Wait for your phone to complete the scan."

Close-up of the CapTel 880iB display. It highlights the Brailiant device.

Narrator: "Make sure your Brailiant Braille display is highlighted."

Close-up of finger pressing the YES button.

Narrator: "Press the YES button to select."

Narrator: "Wait for the devices to pair."

The man who is sitting uses the Brailiant display.

Narrator: "Your Brailiant should now read Scan for Bluetooth Devices, which means that you can now use the Brailiant to navigate through the phone menu and read captions."

The man who is sitting uses the Brailiant display to perform the actions that the narrator is speaking about each time.

Narrator: "To move down, use the spacebar and dot 4."

Narrator: "To move up, use the spacebar and dot one."

Narrator: "For yes, use the spacebar and dot 8."

Narrator: "For no, use the spacebar and dot 7."

Narrator: "Your CapTel 880iB and Brailiant are now ready to use."

The two men in the kitchen have completed the connection between the CapTel 880iB and the Brailiant display.

Brief fade to black.

"CapTel ® 880iB" and "MAKING A CALL USING HUMANWARE BRAILLIANT ®" are shown on the screen.

Close-up of man sitting at counter with the CapTel 880iB phone next to the Brailiant braille reader. He uses the Brailiant through the following statements by the narrator.

Narrator: "To place a call using the Brailiant Braille reader, with the handset hung up, dial the phone number using your Brailiant braille display's keyboard."

Narrator: "You can use dot 7 to backspace if you make a mistake."

Narrator: "When you're finished, lift the handset and the CapTel phone will dial the number of the person you're trying to reach."

Close-up of the CapTel 880iB as it dials and connects.

Woman in an office with headset microphone answers the call.

Receptionist: “Good afternoon, Dr. Cooper's office.”

Close of CapTel 880iB with receptionist's words.

Man seated by himself in a kitchen reads the Brailiant display.

Scott says: “Hi, this is Scott calling and I am wondering if I can set up an appointment to see Dr. Cooper sometime this week.”

Receptionist responses: “Okay, let me see what we have available.”

The receptionist's words appear on the CapTel 880iB screen.

Scott reads the receptionist's words on the Brailiant display.

Receptionist: “It looks like the doctor has an opening at two o'clock on Thursday would that work for you?”

Close-up of Scott's hands on the Brailiant display as he reads.

Narrator: “Navigate through captions at your own pace using space dot 4, space, dot 1, and space.”

Scott: “Actually, yes, that would be great.”

Receptionist answers: “Okay great. We'll see you then. Bye.”

Scott: “Thank you very much. Bye.”

Scott hangs up CapTel 880iB and puts hands back onto the Brailiant display.

Narrator: “When you are finished with your call, simply hang up the CapTel handset.”

Fade to black.

End.